

## Instructions for use of the **CleanWND®** cloth

### **CLEANWND® cloth for Debridement and Cleaning of Complex Wounds.**

Product Code 2015A, Per package: 1 cloth, per box 10 cloths.

#### **Usage of the CLEANWND® Cloth:**

Open the packaging where indicated and take the cloth out of the packaging. Use the cloth to perform debridement and to clean the wound.

The **CLEANWND®** cloth doesn't have a top or bottom side.

Use one **CLEANWND®** cloth per wound to perform debridement and to clean the wound and/or wound surroundings.

Perform (several times) firm rotating movements with the **CLEANWND®** cloth in the wound. Repeat this until (most of) the debris in the wound is removed and you have a good red-blooded wound bed.

The **CLEANWND®** cloth can be cut in two parts, one part to perform debridement of the wound and the other part to clean the surrounding skin. The **CLEANWND®** cloth also treats keratoses of the surrounding skin very effectively!

The **CLEANWND®** cloth is already moist and because of that there are no other fluids needed to debride & clean the wound and surrounding skin.

After usage throw the cloth away as clinical waste.

#### **Debridement & Cleaning of:**

Complex Wounds, like diabetic foot ulcers, venous leg ulcers and pressure ulcers. **CLEANWND®** can be used as a primary dressing. The product is not intended to use on large wounds.

#### **Active ingredients:**

Sodium hyaluronate, Phospholipid, Aloe Vera

#### **Warning:**

Don't use the cloth when the packaging is already open. Usage for babies of 0-6 months old and women that are pregnant or breast-feeding, only under supervision of a doctor.

Patients that are allergic for one of the ingredients of the **CLEANWND®** cloth should not use this product. In case of an allergic reaction, stop immediately with the usage of the cloth and contact your doctor.