

MULTIDEX[®]

An effective treatment for complex wounds!



MULTIDEX[®]

An effective treatment for complex wounds!

WHAT IS MULTIDEX[®] WOUND DRESSING?

Multidex[®] is a wound dressing in gel or powder form, based on Maltodextrin and Vitamin C, for the treatment of complex wounds.

GEL



Moist – dry wound

POWDER



Wet wound

HOW IT WORKS

Multidex[®] creates an optimal moist environment which prevents the wound from drying out and is beneficial for granulation tissue growth and epithelial proliferation.

Maltodextrin stimulates autolytic debridement by attracting white blood cells which digest debris and bacteria through phagocytosis. Maltodextrin also attracts fibroblasts that form collagen and promote the growth of granulation tissue.

Vitamin C creates an acidic environment in the wound by lowering the pH to 4.2. This supports the bactericidal and bacteriostatic mechanism for some aerobic/anaerobic, gram negative/gram positive bacteria. Vitamin C is also necessary for collagen synthesis.

COMPOSITION

Multidex[®] contains:

Maltodextrin NF (D-Glucose Polysaccharide)

1% Ascorbic acid (Vitamin C)

Multidex[®] gel also contains water and glycerin.

FEATURES / BENEFITS

For use on complex, including infected, superficial to deep, dry to moist wounds.

Contributes significantly to autolytic debridement.

Supports significant reduction of (intensely) scented wounds <24-72 hours, and supports odor control.

Mixes rapidly with exudate and penetrates all wound irregularities to fill tunneling and undermining.

Creates and maintains an ideal moisture balance.

No systemic absorption, therefore also suitable for diabetic patients.

Non-toxic, natural ingredients.

Affordable and user-friendly.

INDICATIONS

Venous Leg Ulcers

Diabetic Foot Ulcers

Infected wounds

Superficial / Deep wounds

Pressure sores

Second degree burns

DRESSING CHANGE FREQUENCY

3 times a week

MULTIDEX®

An effective treatment for complex wounds!



PACKING UNIT

Multidex® is available as a Gel (for lightly to moderately exuding wounds) and as a Powder (for moderately to heavily exuding wounds).

ARTICLE CODE	DESCRIPTION	CONTENT	UNIT
46-701NL	Multidex® Powder Wound Filler	45 gram	1 Tube
46-703NL	Multidex® Powder Wound Filler	12 gram	5 Tubes
46-712NL	Multidex® Gel Wound Filler	85 gram	1 Tube
46-710NL	Multidex® Gel Wound Filler	14 gram / 15 ml	1 Box / 10 pieces

PROTEX HEALTHCARE,
Duwijkstraat 17, 2500 Lier, Belgium
info@protexhealthcare.com, www.protexhealthcare.com

Multidex® is a DeRoyal® product



Improving Care. Improving Business.®



Multidex are trademarks or registered trademarks of DeRoyal Industries, Inc. Multidex® is a registered trademark of DeRoyal Industries, Inc. Copyright© 2019, DeRoyal. All rights reserved. All products shown are legally marketed in accordance with United States laws and regulations but may not be available for sale in all countries.